TECHNIQUE GUIDE: FibreKleer™ 4x™Fiber Posts Color Removal

FibreKleer^m 4x^m drills and posts are color coded to streamline the size selection process. While it is not necessary, there may be times when you prefer to remove the paint from the post, after size selection.



 Select the size FibreKleer[™] 4x[™] Fiber Post you wish to place.



2. Using a fine, white rubber wheel, apply light pressure to the painted surface of the post for approximately 10-20 seconds, until the color is removed.



 Clean the post with an alcohol wipe and dry it with a blast of oil-free air. Place the post according to the FibreKleer[™] 4x[™] Fiber Posts Instructions for use.*

PENTRON

