Hot/Cold pack

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En- Reusable thermal cushion (11x12 cm)

Indications – Cold use: Traumatic pain 24-48 hours after the trauma. Post-operative pain 24-48 hours after surgery. Post-operative inflammation 24-48 hours after surgery. Dental pain. **Hot use:** Traumatic pain during scarring. Post-operative pain during scarring. Pain in the temporomandibular joint.

Clinical benefit: Relieves pain, reduces inflammation.

Users: The heat cushion can be used for adults and children, unless contraindicated. Do not use for infants.

Contraindications: Cold therapy: Raynaud's phenomenon. Hypersensitivity to cold. Anesthesia or paresthesia of the treated area. **Hot therapy:** Do not use within 48 hours of surgery. Post-operative haemorrhage. Preliminary stage of a soft tissue contusion or sprain. Skin eczema and allergic dermatitis of the treated area. **Hot and cold therapy:** Sensory disorders. Cognitive or communication deficits. Irradiated skin. Fragile skin (in the elderly, in patients on steroid treatment, in cases of scarring).

Precautions for use: For external use only. Cold therapy should not be used on an area where the nerves run close to the surface. Do not sit, lie or lean on the cushion. Use the cushion under adult supervision. Overheating or prolonged application can cause burns. Do not open the cushion.

Instructions for use: 1-Cold: Put the cushion in the freezer for 2 hours before using it, or keep a cushion in the refrigerator at all times. Then wrap the cushion in the cover provided for this purpose apply it to the affected area.

2-Hot: 2a-Heating in a microwave oven: Put the cushion in the microwave oven, choose a high power level. As microwave ovens vary, it may take a longer or shorter time to heat to the correct temperature. Then wrap the cushion in the cover provided for this purpose and apply it to the affected area. **2b-Heating with hot water:** Heat water in a container until it starts to boil. Remove the container from the heat source. Immerse the cushion in the hot water for 8-10 minutes. Remove the cushion from the water. Check the temperature, if the cushion is too hot, let it cool down a little. When the desired temperature is reached, put the cushion back in its cover. Apply it to the affected area.

Cleaning and reuse: The heat cushion is reusable. Wash it in hot water using dishwashing liquid if necessary, then dry the water with a clean towel before and after use. If the cushion is damaged, dispose of it in the normal household waste system.

Composition: Water, glycerine, sodium carboxymethylcellulose, colouring agent.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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