Instant Freeze

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En- Single-use instant cold pack

Indications: Traumatic pain 24-48 hours after the trauma. Post-operative pain 24-48 hours after surgery. Post-operative inflammation 24-48 hours after surgery. Dental pain. Clinical benefit: Relieves pain, reduces inflammation.

Users: INSTANT FREEZE can be used for adults and children unless contraindicated. Do not use for infants.

Contraindications: Raynaud's phenomenon. Hypersensitivity to cold. Anesthesia or paresthesia of the treated area. Sensory disorders. Cognitive or communication deficits. Irradiated skin. Fragile skin (in the elderly, in patients on steroid treatment, in cases of scarring). High blood pressure. Cardiovascular diseases.

Precautions for use: For external use only. Cold therapy should not be used on an area where the nerves run close to the surface. Do not sit or lie on the cushion. Single use product. Do not reuse, refresh or freeze, as this may cause frostbite. Use the cushion under adult supervision. Contents are harmful if swallowed. If accidentally swallowed, seek medical attention or contact the nearest hospital immediately. Do not pierce the pack. If the pack is pierced and the contents come into contact with the eyes or skin, wash and rinse with plenty of water and seek medical advice.

Instructions for use: 1/ Hold the pack upright and press firmly on the centre with both thumbs to exert sufficient pressure to pierce the bubble inside. 2/ Shake to mix contents. 3/ Wrap the pack in a cloth and apply it to the painful area.

After use, dispose of the product in the usual household waste disposal system.

Composition: Urea, water.

Storage: Store the product in a dry place at room temperature away from sunlight. Storage at lower temperatures may cause frostbite after activation. Keep away from heat and flammable materials. Do not store with combustible materials.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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