Multiply your ability with Sonic energy.

Enjoy the SonicFill[™]3 power!



SonicFill™3 SingleFill™Composite System

Step 1 ApplySonicFill™3 and sculpt. Step 2 There is no step two.

Introducing the new, advanced SonicFill 3 SingleFill[™] Composite System. It's the industry's first sonic-activated bulk fill composite system that helps you craft quality restorations with greater ease and speed compared to incremental layering technique. Achieve long-lasting restorations in one efficient step.



Backed by 125 years of expertise

For the past 10 years, Kerr's SonicFill System has enabled thousands of clinicians worldwide to perform quality posterior restorations with an easy-to-use, SingleFill procedure that combines the advantages of a flowable and a sculptable composite.


The key to its success has been the unique sonic-activated technology, which combines Kerr's exclusive knowledge of material science with the innovative use of Sonic energy, coming from KaVo's know-how in handpieces. The result is a revolutionary product which helps clinicians in reducing chair time, while improving clinical outcomes and patient experience.



Unique, patented technology

SonicFill's secret lays in its highly-filled proprietary resin with special modifiers that react to sonic energy applied through the handpiece.


When sonic energy is applied the viscosity drops (up to 84 %), enabling quick placement and precise adaptation to the cavity walls. When the sonic energy is stopped, the composite returns to a more viscous, non-slumping state that is perfect for carving and contouring.

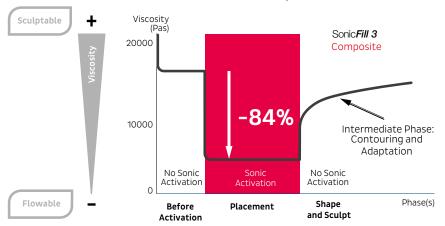

Activation - Efficient Liquefaction

The best benefits of both flowable and sculptable composite in one

New SonicFill[™] 3 is the only Sonic Activated bulkfill composite that acts as a flowable in terms of adaptation and quick placement, but also as a sculptable composite when it comes to carving and contourina.

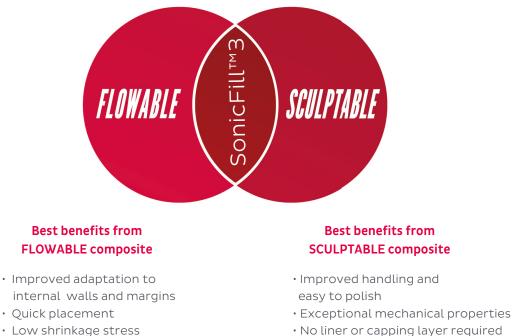

• No liner or capping layer required

The fewer steps, the less chance of misstep

SonicFill[™] 3 replaces conventional time-consuming, multi-stage layering techniques with a SingleFill[™] Composite System that eliminates the need for placing a liner or final capping layer.



Traditional layering technique. Several Layers.



Bulk fill flowable with universal cap. Two Layers.

High viscosity bulk fill restorative with flowable liner. Two Layers.




SonicFill[™] System UP TO 5 MM ONLY 1 LAYER

Proven outcomes. A step up from the rest

Shrinkage Stress

SonicFill[™] 3 has a lower shrinkage stress than competitors.*

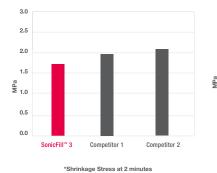

Flexural Strength

165

160

155

150

145

140

135

130

125

120


115

SonicFill[™] 3

SonicFill™ 3 has a higher flexural strength than competitors.*

Degree of Conversion

At 5mm, SonicFill™ 3 showed 86.6% hardness at the bottom when compared to the top.*

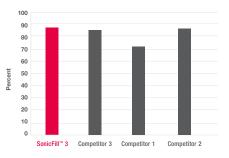

* Kerr Internal Test Data Data on file, available upon request

Quality restorations that stand the test of time, in less time

Competitor 3


Competitor 1

Competitor 2

Crafting long-lasting restorations has never been easier or more efficient. The new, advanced SonicFill[™]3 with its improved adaptation is already helping dentists perform restorative work with greater ease and accuracy, and with more predictable and reliable outcomes. SonicFill[™]3 is indicated for direct placement in all cavity classes in anterior and posterior teeth. It's also indicated for minor occlusal build-ups and core build-up (Refer to the IFU for a complete list of indications).

Old amalgam replacement with SF3



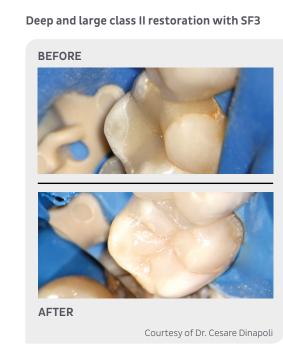

Take advantage of the SonicFill™3 power! 3 steps forward in performance


SonicFill[™] 3 bulk fill composite system transforms tedious, repetitive posterior restorations into an easy and reliable Single-Fill placement. The new advanced SonicFill™ 3 bulk fill features improved adaptation, it is even easier to handle and sculpt, while also being simpler to extrude during placement, as compared to its predecessors. The end benefits are faster patient visits, more predictable results, and dependable, long-lasting restorations.





Confident execution

The dramatic reaction to the sonic energy applied through the exclusive KaVo handpiece reduces the viscosity of the Kerr nanohybrid composite up to 84%. This allows great adaptation, for reliable and predictable restorations.

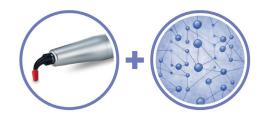

BE FAST

Shorter procedure

SonicFill 3 transforms tedious, repetitive posterior restorations into an easy and fast Single-Fill placement: filling cavities up to 5mm in depth in a single increment, with a single material - and no liner or capping layer. Ergonomic tips allow easy access to the cavity.



BE SAFE



Lasting performance

High filler loading (81% by weight).

Lower shrinkage stress allows great marginal integrity, minimizing voids and gap in the composite; on top of this excellent mechanical properties, such as excellent wear resistance and high flexural strength, ensure strength and durabilty over time.



Stay tuned

Versatile products, versatile communication. There are many ways to keep in touch with Kerr Dental and learn about our latest news and developments.

Receive updates

Discover what's new

F D in O Connect with us

Experience the SonicFill™3 SingleFill composities system in your practice	te

Talk to us

Ordering information

SonicFill™3 Intro Kit

The Intro Kit is an ideal way to get introduced to SonicFill 3. It contains a SonicFill Handpiece, a "Getting Started Kit", a Technique Guide, and a total of 60 unidose tips: 30 Unidose tips in A3, 20 Unidose tips in A2 and 10 Unidose tips in A1.

36715 SonicFill™3 Intro kit

SonicFill™3 unidose refills

Each refill contains 20 unidose tips each 0.25gr

- 36711 SonicFill™3 composite A1
- 36712 SonicFill™3 composite A2
- 36713 SonicFill™3 composite A3
- 36714 SonicFill™3 composite B1

Handpiece and accessories

1.007.7400	SonicFill™ Handpiece
0.553.1210	MultiFlex Coupling 457



