

ImpressPLUS Bite Registration

Addition Curing Silicone Bite Registration Material

Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by a professional only.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Product Name	Impress Plus Bite Registration Material
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - REF

PP010/1350 – ImpressPlus Bite Reg Regular set

PP010/1352 – ImpressPlus Bite Reg Fast Set

PP010/1354 – ImpressPlus Bite Reg Super-Fast Set

Indications	<ul style="list-style-type: none"> • Accurate occlusal registrations • Key material for needle point registration • Production of small model segments 		
	Super-Fast Set	Fast Set	Regular Set
Total working time*	0:30	0:40	1:30
Time in mouth	0:45	1:00	3:00
Total setting time*	1:15	1:40	4:30
Dimensional change (%)	<0.1		
Shore A hardness	85		

*Application is at 23°C ± 2 / 73°F ± 4, 50 ± 5% relative humidity. **Attention!** Higher temperature reduces working and setting times and lower temperature prolongs them.

INSTRUCTIONS

Mounting the cartridge: Insert cartridge onto dispensing gun following the manufacturer's instructions and remove cap. Extrude impression material until it exits both chambers at the same rate and then mount the appropriate mixing tip. An intra-oral tip may be placed on the mixing tip.

APPLICATION

Apply bite registration material according to the desired technique (occlusal or buccal). Extrude material first onto areas where teeth have been prepared keeping the end of the mixing tip immersed in the material to avoid the formation of air bubbles and ask the patient to bite. Carefully remove the set bite registration after the recommended time. Remove all residues of the material from the mouth. Trim bite registration to ensure correct seating on model. After use keep mixing tip on cartridge as it serves as a cap. Intraoral tips and mixing tips are for single use only.

DISINFECTING

Once the bite registration is removed from mouth, rinse under cold water. The impression can then be disinfected with appropriate disinfecting solutions or sprays such as 2% glutaraldehyde. Take care to follow the manufacturer's instructions for use. For further information regarding disinfecting impressions: JADA 1991; vol. 122; IS(8); p110.

PRECAUTIONS

- Contact with latex gloves may impair the setting reaction. Vinyl gloves are recommended.
- Do not combine with condensation curing silicones, polyether or polysulfide.
- Avoid contact of material with eyes. In case of contact rinse eyes with plenty of water and consult a physician immediately.
- Avoid spillage of material on clothing as it cannot be removed chemically.
- The substances that the product contains may cause allergic reactions to certain individuals.
- Do not use after the expiry date.

STORAGE

Store material between 10 - 25°C (50 - 77°F).

Store impression at room temperature (max. 25°C/77°F).



WARNING

These materials have been developed for use exclusively by dentists and must be handled according to the instructions of use. The recommendations included in the instructions correspond to the actual state of knowledge regarding dental techniques. The manufacturer cannot be held responsible for damages caused by other uses or incorrect handling.

Caution:



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

LOT NUMBER AND EXPIRY DATE

The above are printed on each package and should be quoted in all correspondence which requires identification of the product.



Perfection Plus Ltd
6 Westwood Court
Brunel Road
Hampshire
SO40 3WX, UK
www.perfectionplus.com



Perfection Plus EU Ltd
The Black Church
St Mary's Place.
Dublin, Ireland,
D07 P4AX
Regulatory@perfectionplus.com



Advena Ltd
Tower Business
Centre
2nd Floor Tower
Street
Swatar, Malta

